

STERI*SIMPLE*TM
Reprocessing. Simplified.

Configuration of STATIM Autoclaves

How to Configure your STATIM Autoclave for use with STERIREADER

This manual is intended for use with the “old” beige STATIM autoclaves. This applies to both STATIM 2000 and STATIM 5000 models. There is an accompanying video on YouTube which you may find very helpful.


There are three settings that need to be configured for proper operation with STERIREADER cycle logging and label production software.

- A. Selection of Serial Printer.
- B. Setting of End of Line character.
- C. Setting of Degree symbol.


Each section accesses the User Setup menu by powering up while holding the STOP button on the keypad. It is best to perform all three settings consecutively, but if you need to, each section can be done separately.

A. Selection of Serial Printer


1. Begin by powering down your STATIM.
2. Access the User Setup menu by powering up the unit while pressing the STOP  button. The display screen will show the Time/Date Setup.

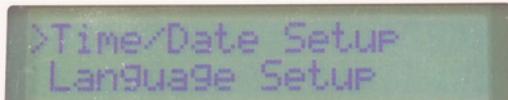

3. Press UNWRAPPED  5x to bring you to the RS232 screen. Press PLASTICS  1x to accept.
4. Press UNWRAPPED  until you see Serial Printer. Press PLASTICS  1x to accept.

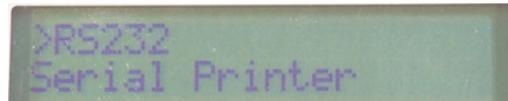


5. This will return you to the RS232 screen and you're ready to set the End of Line character in Part B.

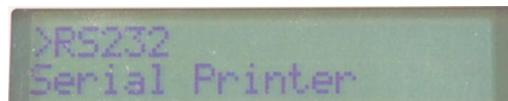


B. Setting of End of Line character.


1. You should be at this screen.

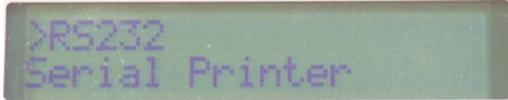

2. Press UNWRAPPED  1x to bring you to this screen. Press PLASTICS  1x to accept.

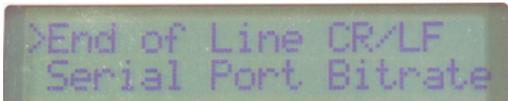

3. Press UNWRAPPED  until you see CR/LF. Press PLASTICS  1x to accept.

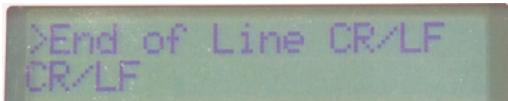


4. This will return you to this screen and you're ready to set the Degree symbol.




B. Setting of Degree symbol.

1. You should be at this screen.

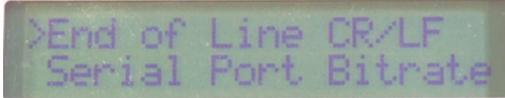

2. Press UNWRAPPED  2x to bring you to this screen. Press PLASTICS  1x to accept.

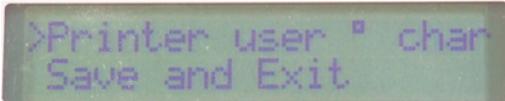


3. You have to advance until you find the "32 [0 x 20]" selection.


Pressing UNWRAPPED  advances by one selection,

Pressing WRAPPED  advances by 10 selections.

4. When you've found this screen, press PLASTICS  1x to accept.

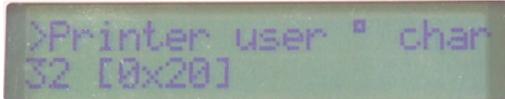

5. Press UNWRAPPED  1x to bring you to the Save and Exit screen. Press PLASTICS  1x to save.

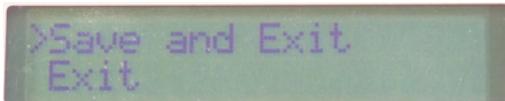

6. The settings are complete! The statim will click and you will see the screen below. It is ready for use with STERIREADER.



Thank you for using STERIREADER
and for your support of the STERISIMPLE System.